Kanso[®] Halo Accessory

User Guide

EN

EL

ES

PT

PT-BR

RO

HE

Contents

| | | |
|--------------|--|-----------|
| EN | Kanso® Halo Accessory User Guide | 1 |
| EL | Πρόσθετο εξάρτημα Kanso® Halo Οδηγός χρήστη | 20 |
| ES | Accesorio Kanso® Halo Guía de usuario | 39 |
| PT | Guia do utilizador do Acessório Kanso® Halo | 58 |
| PT-BR | Guia do usuário do Acessório Kanso® Halo | 77 |
| RO | Accesoriu Kanso® Halo Ghid de utilizare | 96 |
| 115 | שמתשמל ךירדמ Kanso® Halo לש רזיבא | HE |

Contents

| | |
|-------------------------------------|----|
| Introduction | 2 |
| Kanso Halo Accessory. | 3 |
| Use the Halo Accessory | 7 |
| Remove the Halo Accessory | 10 |
| Regular care | 11 |
| Cautions | 12 |
| Warnings. | 13 |
| Other information. | 15 |

Introduction

About this guide

This guide is intended for hearing implant recipients and their carers using the Kanso® Halo Accessory.

Note

- Refer to the relevant sections for cautions and warnings relating to the use of the Halo Accessory.
- Please also refer to your **Important Information** document for essential advice that applies to Cochlear implant systems.

Symbols used in this guide

Note

Important information or advice.

Tip

Time saving hint.

Caution (no harm)


Special care to be taken to ensure safety and effectiveness. Could cause damage to equipment.

Warning (harmful)

Potential safety hazards and serious adverse reactions. Could cause harm to person.

Kanso Halo Accessory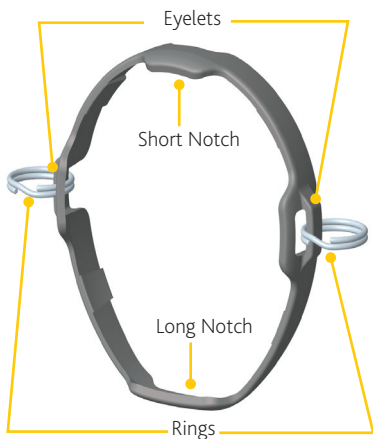

The **Halo Accessory** is a retention accessory for your sound processor. This accessory provides users of the Off-the-Ear (OTE) Kanso 2 Sound Processor increased retention during activity or improved security against dropping the device. The users will require hairclips to use with the Halo Accessory.

**Note**

There is an option to use double-loop safety lines to attach hairclips to the Halo Accessory after removing the rings.

The Halo Accessory should be used by recipients who can either remove the sound processor themselves if it's causing discomfort or who can indicate any discomfort to their parent or caregiver.

Intended purpose

The Halo Accessory is intended to be used to assist with wearing the sound processor as part of a hearing implant system.

Indications

The Halo Accessory is indicated for use with the Kanso 2 Sound Processor.

Contraindications

The Halo Accessory is not indicated for use with sound processors other than those listed in Indications.

Intended patient population

The sound processor is intended for patients implanted with a compatible Cochlear Nucleus implant. There are no restrictions for the intended patient population of the sound processor in terms of age, weight, health or other condition.

Benefits

The sound processor operates in conjunction with a compatible Cochlear Nucleus implant system.

Potential benefits of receiving a Cochlear Nucleus implant system include:

- Better understanding of speech in quiet.
- Better understanding of speech in noise.
- Increased satisfaction based on hearing capabilities.


Intended users

The devices listed in Indications are intended for the following users:

- Recipients of a compatible Cochlear Nucleus implant.
- Carers of recipients, who carry out recipient functions as needed. Carers may include parents of paediatric recipients who are less than 12 years old, and nurses or other carers of otherwise dependant recipients.
- Audiologists/clinicians.

Use the Halo Accessory

1. Align the eyelets and notches of the Halo Accessory with the Kanso 2 Sound Processor.
 - i. Shorter notch aligns with the top
 - ii. Longer notch aligns with the base

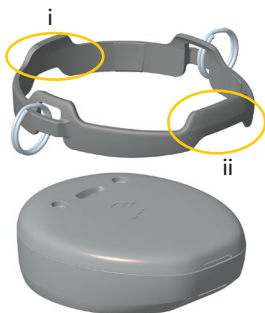
2. Ensure that the Halo Accessory is aligned parallel to the sound processor.


3. Push the Halo Accessory until it slides into place on the sound processor. Ensure the sound processor is secure.

-
4. Thread the hairclips to the rings attached to the Halo Accessory.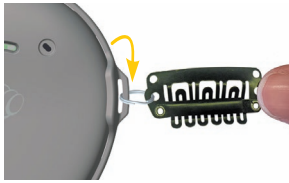

-
5. Mount the sound processor on to the implant, with hairclips hanging from the rings.

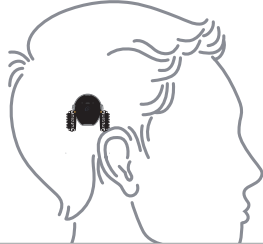
-
6. Attach the hairclips to hair on either side of the sound processor.

-
- a. Hold the hairclip from the edges and press in the middle to open it.


- b. Thread the hair through the hairclip.

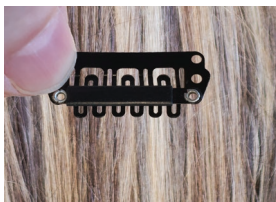- c. Press the edges of the hairclip to close.

7. Check if the hairclips are secure and orient in V-shape with the sound processor.

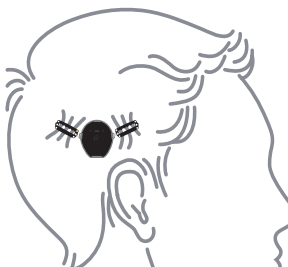 **Caution**

To avoid damage to your sound processor, make sure the Halo Accessory is properly secured.

 **Note**

When the Halo Accessory is on your sound processor, a portable charger can be attached.

Remove the Halo Accessory

Caution

Always remove the Halo Accessory before charging your sound processor in the Home Charger. If the Halo Accessory is attached to your sound processor in the Home Charger, charging could be slow or may not work.

1. Detach hairclips from your hair.
2. Remove your sound processor from the implant.
3. Support the Halo Accessory on either side with your fingers.
4. Push the sound processor through with your thumbs.

Regular care

Caution

Do not use cleaning agents or alcohol to clean your processor or Halo Accessory.

Caution

To avoid deformation or breakage of your Halo Accessory, do not wring or twist it.

- Check the Halo Accessory for dirt or moisture regularly. Wipe the Halo Accessory with a soft dry cloth.
- Check if the Halo Accessory is becoming loose or showing signs of wear. Replace as needed.

Cautions

- The Halo Accessory is a retention device. Do not use it for any other purpose.
- Always use the Halo Accessory with a compatible sound processor.
- If your Halo Accessory appears damaged or worn, replace it with a new one.

Warnings

- If your sound processor becomes warmer than usual, remove it immediately and seek advice from your clinician. Parents/caregivers: if your recipient indicates discomfort, check if their sound processor is warmer than usual.
- Sound processors and related accessories contain small parts that alone or in combination may pose a hazard of inhalation, choking or ingestion. Swallowing or inhaling small parts can cause severe or fatal injuries. Use the tamper-resistant locks to keep small parts attached to the processing unit where available. Always supervise children under 3 years of age and others who may be at risk of inhalation, choking or ingestion of small parts when they use the sound processor and related accessories. When not in use, keep small parts, and combinations of small parts, out of reach from children. If small parts are swallowed or inhaled, seek immediate medical attention.
- The Halo Accessory and sound processor should be protected from excessive heat sources such as prolonged exposure to bright sunlight. Before wearing the Halo Accessory and sound processor, check the surface to ensure it is not too hot to wear without discomfort to skin.
- Exposure of the sound processor and Halo Accessory to temperatures above the operating range (+5° C/+41° F to +40° C/+104° F) could lead to increased risk of sound processor shutdown or malfunction due to overheating.

EN

- If you experience tightness or pain at the implant site, or develop significant skin irritation, stop using your sound processor and contact your clinician.
- If the coil magnet is too strong, pressure sores may develop at the implant site. If this happens, or if you experience any discomfort in this area, contact your clinician.
- No modification of this equipment is allowed.
- Use only as intended.
- Report unexpected problems to your clinician.

Other information

| ITEM | MATERIAL |
|----------------|---------------------|
| Halo Accessory | Nylon |
| Rings | 316 Stainless Steel |

Product component measurements

Typical values.

| ITEM | WEIGHT | WIDTH | LENGTH |
|----------------|--------|---------|---------|
| Halo Accessory | 1 g | 38.9 mm | 40.1 mm |

Environmental conditions

| CONDITION | MINIMUM | MAXIMUM |
|---|-----------------|------------------|
| Storage and transport temperature | -10° C (+14° F) | +55° C (+131° F) |
| Storage and transport humidity | 0% RH | 90% RH |
| Operating temperature (sound processor) | +5° C (+41° F) | +40° C (+104° F) |
| Operating relative humidity | 0% RH | 90% RH |
| Operating pressure | 700 hPa | 1060 hPa |

Disposal

Help protect the environment by not disposing of your Halo Accessory with your unsorted household waste. Please recycle your Halo Accessory according to your local regulations.

Warranty

The use of the Halo Accessory will not impact your Nucleus Global Limited Warranty for your sound processor, if used in accordance with the Halo Accessory instructions.

Labelling symbols

The following symbols may appear on your sound processor or accessories and/or packaging:

Consult instructions for use

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

Compatible sound processors

Authorised representative in the European Community

Unique Device Identification


Catalogue number

Medical device

Date of manufacture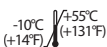

Temperature limits

IP68

Protected against failure from dust penetration

Protected against failure from continuous immersion in water up to 3 metres deep for up to 2 hours

CE registration mark with notified body number

CE registration mark

Recyclable packaging

Serious incidents

Whilst serious incidents in relation to medical devices are rare, it is acknowledged that incidents may happen. As an organisation, Cochlear recognises the potential for harm and will respond to any reported serious incident.

What is a serious incident?

A 'serious incident' means any event that directly or indirectly has caused or could have caused an unexpected or unwanted event including any of the following:

- The death of a patient, user or other person,
- The temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- A serious public health threat.

Reporting a serious incident

There is no definitive list of events/incidents that constitute a serious incident, however all serious incidents should be reported to:

- your local Cochlear office
www.cochlear.com/intl/contact/global-offices
- For users within the European Union, you should also report serious incidents to your National Competent Authority, http://ec.europa.eu/growth/sectors/medical-devices/contacts_en
- For users outside of the European Union, if applicable, report serious incidents to your National Competent Authority,
- For users within Australia, report serious incidents to the Therapeutic Goods Administration <https://www.tga.gov.au>

Legal statement

The statements made in this guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

© Cochlear Limited 2021

Certification

The CE mark applies to the Kanso Halo Accessory.

Trademark legal notice

Cochlear implant systems are protected by one or more international patents. ACE, Advance Off-Stylet, AOS, AutoNRT, Autosensitivity, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントゥア, Contour Advance, Custom Sound, ESPrit, Freedom, Hear now. And always, Hugfit, Hybrid, Invisible Hearing, Kanso, MET, MicroDrive, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Profile, Slimline, SmartSound, Softip, SPrint, True Wireless, the elliptical logo, and Whisper are either trademarks or registered trademarks of Cochlear Limited. Ardium, Baha, Baha SoftWear, BCDrive, DermaLock, Human Design, Piezo Power, SoundArc, Vistafix, and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB.

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Cochlear Americas 10350 Park Meadows Drive, Lone Tree, CO 80124, USA

Tel: +1 303 790 9010 Fax: +1 303 792 9025

Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada

Tel: +1 416 972 5082 Fax: +1 416 972 5083

Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland

Tel: +41 61 205 8204 Fax: +41 61 205 8205

Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom

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Tel: +90 216 538 5900 Fax: +90 216 538 5919

Cochlear (HK) Limited Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong

Tel: +852 2530 5773 Fax: +852 2530 5183

Cochlear Korea Ltd 2nd Floor, Yongsan Centreville Asterium, 25, Hangang-daero 30 gil,

Yongsan-gu, Seoul, Korea (04386)

Cochlear Medical Device (Beijing) Co., Ltd

Unit 2608-2617, 26th Floor, No.9 Building, No.91 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China

Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

Cochlear Medical Device Company India Pvt. Ltd.

Ground Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India

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Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

Cochlear Middle East FZ-LLC

Dubai Healthcare City, Al Razi Building 64, Block A, Ground Floor, Offices IR1 and IR2, Dubai, United Arab Emirates

Tel: +971 4 818 4400 Fax: +971 4 361 8925

Cochlear Latinoamérica S.A.

International Business Park, Building 3835, Office 403, Panama Pacifico, Panama

Tel: +507 830 6220 Fax: +507 830 6218

Cochlear NZ Limited

Level 4, Takapuna Towers, 19-21 Como St, Takapuna, Auckland 0622, New Zealand

Tel: + 64 9 914 1983 Fax: 0800 886 036

www.cochlear.com

D1941568 V1

Multilingual version of P1944412-D1895506 V1 2022-01

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Cochlear Nordic AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden
Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.